


REPLACING THE CARBON HEEL ON THE DURALITE™ FOOT

1. Using a 5mm hex wrench, remove the two M8 cap screws as shown.
2. Remove the heel.
3. Insert the new heel.
4. Apply Loctite 242 (or equivalent) to the M8 cap screws.
5. Install the M8 hardened flat washers onto the M8 cap screws.
6. Install the screws with the washers. Tighten the screws to 16 ft-lbs (21 Nm).

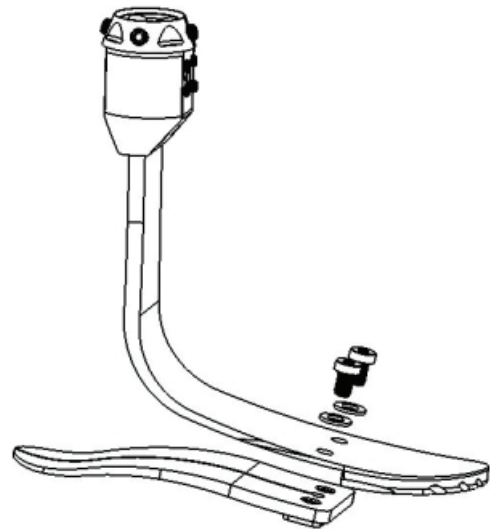

OHIO WILLOW WOOD®

free the body...free the spirit®

15441 Scioto Darby Road

Mt. Sterling, OH 43143


phone 740.869.3377 / 800.848.4930

fax 740.869.4374 www.owwco.com

PN-2037-A 23 AUG 2007

REPLACING THE CARBON HEEL ON THE DURALITE™ FOOT

1. Using a 5mm hex wrench, remove the two M8 cap screws as shown.
2. Remove the heel.
3. Insert the new heel.
4. Apply Loctite 242 (or equivalent) to the M8 cap screws.
5. Install the M8 hardened flat washers onto the M8 cap screws.
6. Install the screws with the washers. Tighten the screws to 16 ft-lbs (21 Nm).

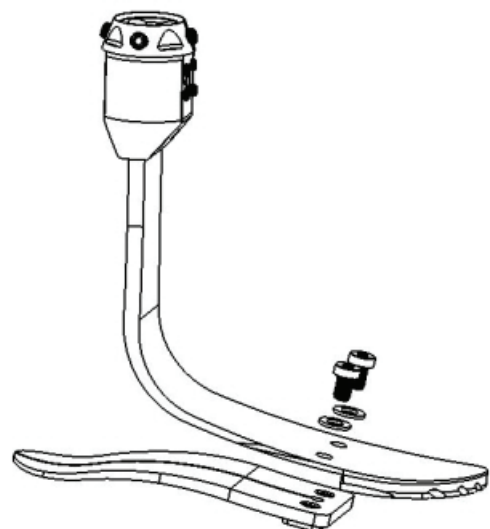

OHIO WILLOW WOOD®

free the body...free the spirit®

15441 Scioto Darby Road

Mt. Sterling, OH 43143

phone 740.869.3377 / 800.848.4930

fax 740.869.4374 www.owwco.com

PN-2037-A 23 AUG 2007