

## IMPORTANT INFORMATION FOR THE PROSTHETIST

### Trimming the Alpha® Spirit Liner:

If trimming of the Alpha® Spirit Liner is desired, make a mark on the Alpha® Spirit Liner to indicate the position of top edge of the thin band of fabric inside the Alpha® Spirit Suction Seal. Follow the trimming instructions in the Alpha® Liner Instruction Manual, and do not trim the Alpha® Spirit Liner below this mark.

### Trimming the Alpha® Spirit Suction Seal:

Although normally not necessary, the Suction Seal may be trimmed to a shorter length if desired. Use large shears, a paper cutter, or a Hand-Held Cutting Wheel (OWW Part No. 700-TL002). Do not cut off the thin band of the fabric near the proximal end of the Suction Seal.

If using the Hand-Held Cutting Wheel, place the Suction Seal on a cutting board or thick piece of plastic. Using a straight edge as a guide, roll the cutting wheel across the Suction Seal.

Be certain that the socket brim has no sharp edges and is covered by leather or alternate padding. Sharp edges on the socket can wear a hole in the Suction Seal.



OHIO WILLOW WOOD®

*free the body...free the spirit®*

15441 Scioto Darby Road

Mt. Sterling, OH 43143

phone 740.869.3377 / 800.848.4930

fax 740.869.4374 [www.owwco.com](http://www.owwco.com)



Ohio Willow Wood Company B.V  
Keizersgracht 62/64  
1015 CS Amsterdam  
The Netherlands



PN-1416-B 8 JUN 2009