

LimbLogic® VS

Patient Information



The LimbLogic VS features an electronically controlled Vacuum Pump that uses air pressure and a sealing sleeve to securely suspend your prosthesis.

The fob included with the system provides you with wireless control and monitoring of the vacuum level.

Please review these instructions thoroughly and contact your prosthetist if you have any questions.

LimbLogic VS - 1600 Series is now waterproof.



OHIO WILLOW WOOD®
free the body...free the spirit®



There are no field-serviceable parts inside the Vacuum Pump. Opening the Vacuum Pump will void the warranty.



This pump is only designed to move air; use of Vaseline® or similar lubricating creams inside the socket will clog the pump. Do not allow foreign substances to be pulled through the Vacuum Pump. This may impair function of your vacuum system.



Do not allow acetone to contact the Vacuum Pump or fob.



The LimbLogic VS is not intended to be used as the only means of attachment of prosthesis to patient.



All LLVS components have passed safety testing for use as medical devices. Radio enabled devices transmit with a power of less than one milliwatt and comply with United States and international guidelines for low power transceivers. If LLVS components will be used around safety critical devices such as pacemakers or defibrillators, consult the manufacturer for appropriate usage instructions. Consult the section on Regulatory Information for more information on safety and compliance.

Table of Contents

	Page
Quick-Start Guide	4
Charging the Vacuum Pump Battery	5
Using the Battery Charger Outside the United States	6
Charging the Vacuum Pump Battery in a Vehicle.....	6
BK Application	7
AK Application	8
Turning on the Vacuum Pump.....	9
Turning Off the Vacuum Pump	9
Turning on the Fob.....	10
Information Screen.....	11
Active Mode	11
For Bilateral Applications	12
Standby Mode.....	12
Detecting a Leak	13
No Radio Connection.....	13
Low Battery Mode	13
Cleaning the System	14
Exposure to Water	14
Replacing the Fob Battery	15
Troubleshooting Guide	16
Regulatory Information.....	19

Quick-Start Guide


1. Turn on the Vacuum Pump.




2. Start regulating the vacuum.




OR

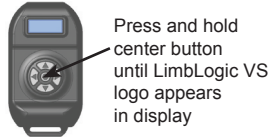

3. Information about the system is displayed.

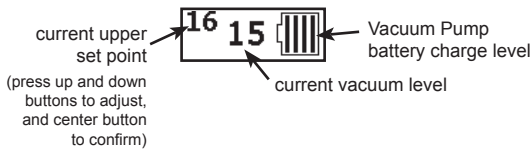

4. To enter Standby Mode:

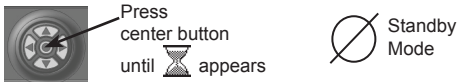

To resume vacuum regulation:






5. Turn off the Vacuum Pump.



Charging the Vacuum Pump Battery

Vacuum Pump

Charger

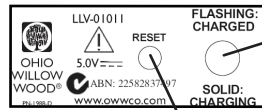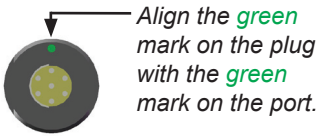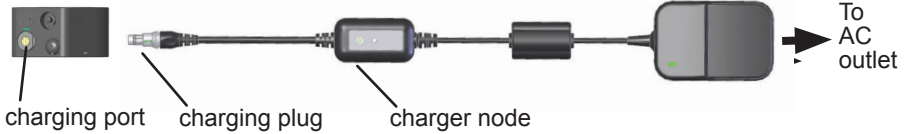

reset button

Green LED
SOLID = charging
FLASHING = charged

Before using the system, charge the Vacuum Pump Battery as follows:

1. Remove the cap from the charging port on the Vacuum Pump.
2. Insert the charging plug into the charging port as follows:
 - a) Gently insert the plug into the port, holding the black plastic portion of the plug, with just enough force to feel the plug touch the port.
 - b) Align the **green** alignment marks on the charging port and the charging plug.



The charging plug can only be inserted into the Vacuum Pump charging port in one orientation. Forcing the plug will damage the plug, will possibly damage the vacuum system, and will void the warranty.

3. Plug the charger into a power outlet. The **blue** light on the two-pronged adapter will light, followed by the **green** LED. The fob will display the **battery charging** indicator.

If the **green** LED does not light, press and hold the **reset button** until the light comes on.

4. When charging is complete, the **green** LED will flash. Disconnect the power cord from the Vacuum Pump and from the power outlet.

5. Reinstall the cap onto the charging port on the Vacuum Pump.



Use ONLY an Ohio Willow Wood charger. For LLV-01042 series Vacuum Pumps use Ohio Willow Wood charger part number LLV-01011. For LLV-01040 series Vacuum Pumps use Ohio Willow Wood charger part number LLV-01010. Use of any other charger may result in damage to the Vacuum Pump or the charger or both.

If the charger needs to be reset, press the **reset button**. With normal usage, the rechargeable battery should last for the life of the product.

Using the Battery Charger Outside the United States

If you have an international charger, your charger will come with a collection of prong adaptors that will allow you to use your charger in many countries.

Prong
Set



Plugging a prong set into an outlet without the charger may expose you to dangerous voltages. Always plug the prong set into the charger before plugging the charger or the prong set into an outlet.



Charging the Vacuum Pump Battery in a Vehicle

To use a vehicle's power to charge the Vacuum Pump, you must purchase a Power Inverter rated at 75 Watts or less to convert the vehicle's 12V DC power to 120V AC. Examples of acceptable Power Inverters include the Digital Concepts™ 75 Watt Portable Power Inverter (available from general retailers) and the Enercell™ 75 Watt Power Inverter (available from Radio Shack®; item number 22-133).

1. Plug the Power Inverter into the vehicle's cigarette lighter socket.
2. Connect the charger to the Vacuum Pump.
3. Plug the charger's two-pronged adapter into the Power Inverter's three-pronged receptacle.

BK Application

After charging the Vacuum Pump battery, don an Alpha Liner and **Alpha Flex Sleeve** as shown below.

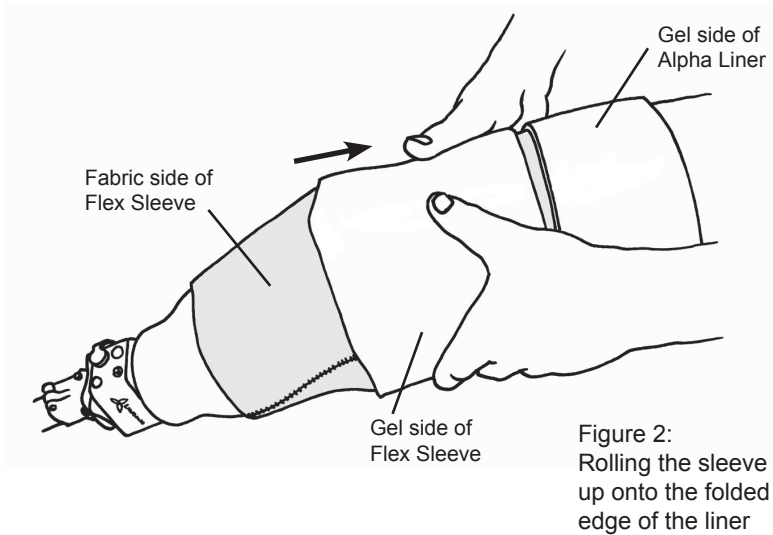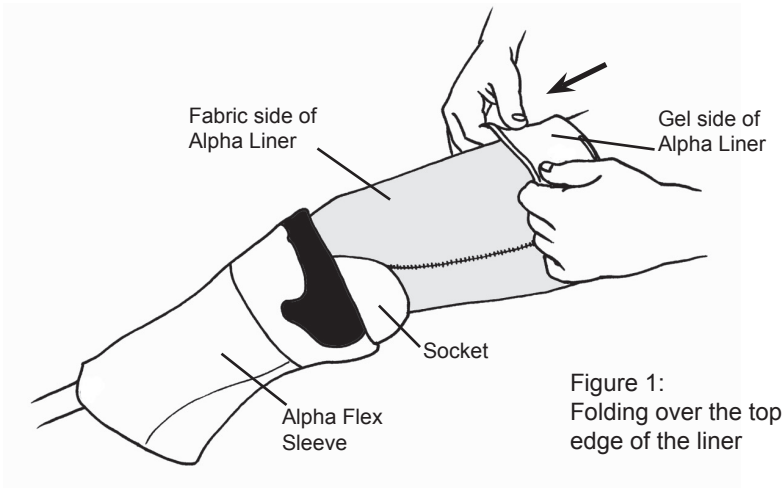

AK Application

After charging the Vacuum Pump battery, don an Alpha Liner and **Alpha AK Sleeve** as shown below.

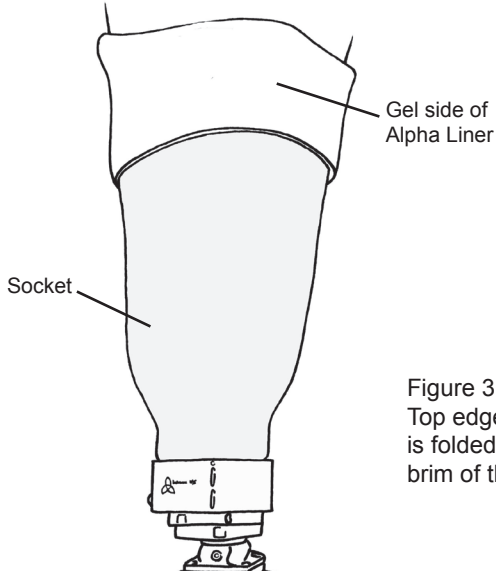


Figure 3:
Top edge of the liner
is folded over the
brim of the socket


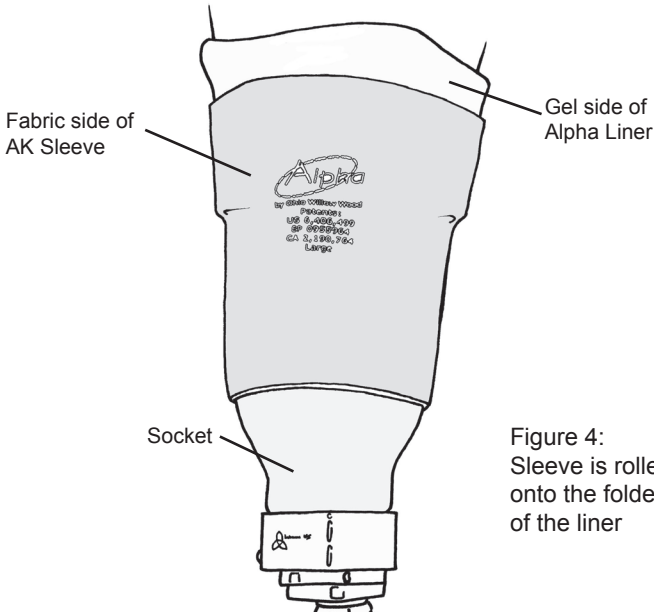


Figure 4:
Sleeve is rolled up
onto the folded edge
of the liner

Turning on the Vacuum Pump

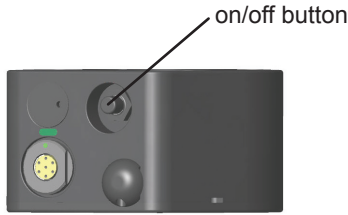

Press the **on/off button** on the Vacuum Pump one time. The pump will vibrate four times.

Note: if the pump has just been unplugged from the charger, the button does not need to be pressed.


When you are ready for the pump to begin regulating the vacuum, press the **on/off button** again (or turn on the fob, as described on the next page). The pump will operate until the preselected vacuum level is reached.

Turning off the Vacuum Pump

Press the **on/off button** on the Vacuum Pump two times. The pump will vibrate twice.

If the current vacuum level is near the current Upper Set Point (refer to page 11), the Vacuum Pump may skip the vibrations to prevent the vacuum level from exceeding the Upper Set Point.

Turning on the Fob

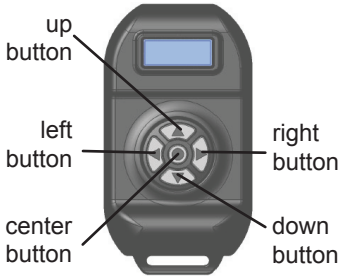

Up/down buttons:
adjust vacuum setting

Center button:
apply setting

Right/left buttons:
toggle between right
and left Vacuum Pumps
(bilateral only)

Press and hold the **center button** on the fob until the **LimbLogic VS** logo appears and the display has reached the desired brightness.

Release the button. If the Vacuum Pump is charged and powered, the **Information Screen** will be displayed.

The fob shuts off after one minute of non-use. This will not affect the Vacuum Pump. Press and hold the **center button** to restart the fob.

Information Screen

The Information Screen indicates important details about the operation of the system.




The vertical bars inside the battery symbol on the right side of the screen indicate the amount of charge left in the Vacuum Pump battery.

As the battery is discharged, the bars disappear from left to right.

a) Active Mode

When numbers are displayed to the left of the battery symbol, as shown below, the system is in **Active Mode**. In this mode, the Vacuum Pump activates as needed to maintain the vacuum settings.

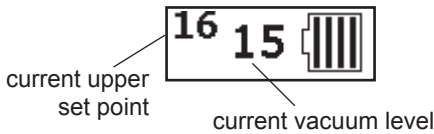

You may use the **up/down buttons** to adjust the Upper Set Point as desired.




Press the **center button** to save the change. The hourglass symbol will appear to indicate that the setting is being saved.

For bilateral applications:


The Information Screen displays right and left indicators to show which pump is currently communicating with the fob.

Use the **right/left buttons** to select the right or left Vacuum Pump, then press the **center button** to save your selection.

The **LimbLogic VS** logo will appear, followed by the **Information Screen** for the selected pump.

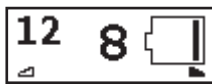

In this example, the fob is communicating with the **RIGHT** pump

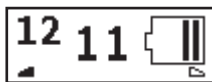

In this example, the fob is communicating with the **LEFT** pump

b) Standby Mode

Allows you to turn off the pump temporarily to prevent the pump from regulating the vacuum.



To enter **Standby Mode**, press and hold the **center button** until the hourglass symbol appears.



The **standby indicator** will then appear to indicate that the pump is not regulating.



To return to **Active Mode**, press and hold the **center button** until the hourglass symbol appears.

After six hours of Standby Mode without radio communication, the unit will turn off. To turn the pump back on, either press the on/off button on the Vacuum Pump one time or plug in the charger.



Keep in mind that a Vacuum Pump in Standby Mode will continue to use the pump battery.

Detecting a Leak



If the leak indicator appears **when you initially don your prosthesis**, restart the pump.

If the leak indicator appears frequently **while you are walking**, contact your prosthetist.

To resume automatic vacuum regulation after a leak has been detected:

- Press the **center button** on the fob until the hourglass symbol appears,
or
- Press the **on/off button** on the Vacuum Pump one time.

No Radio Connection



If the **No Radio Connection** indicator appears:

- make sure the Vacuum Pump has been turned on
- make sure the Vacuum Pump battery has been charged
- make sure the fob is within 3 feet of the Vacuum Pump
- make sure the fob is more than 3 inches away from the Vacuum Pump

If the **No Radio Connection** indicator still appears, contact Ohio Willow Wood.

Low-Battery Mode

If the Vacuum Pump battery has less than 15% of its capacity remaining, the Vacuum Pump enters a “stutter” mode. (Stutter mode is when the pump quickly turns on and off when pulling vacuum.) Charge as soon as possible.




The fob will display the **battery charging** indicator when the unit is plugged into the charger.

Cleaning the System

Once a month, the patient should pour 1/8 cup of **distilled** water or rubbing alcohol into the socket, place the end of the exhaust port tubing into a cup, and activate the Vacuum Pump. Use **ONLY distilled water or rubbing alcohol to clean the system.**



Do not use acetone or similar chemicals to clean the system.

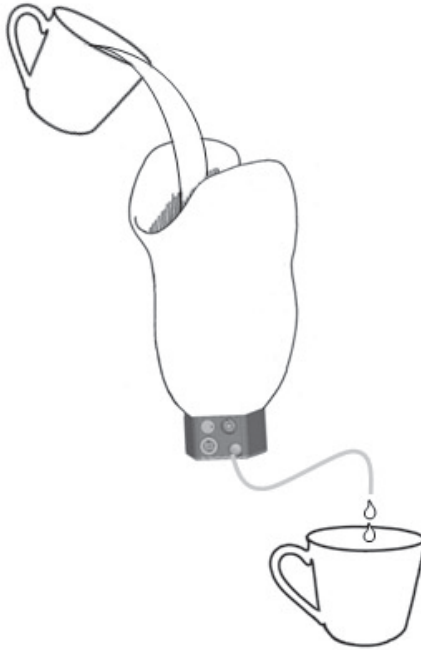

Exposure to Water

The Vacuum Pump is waterproof in fresh water up to a depth of 10 feet or 3 meters for up to 12 hours. The Vacuum Pump should not be submersed in salt water. Be sure to dry off the Vacuum Pump if it is exposed to rain or other moisture.

The fob is not water resistant. Submersion of the fob will void the warranty.


Replacing the Fob Battery

If the fob screen flashes off quickly after you turn it on, then the fob battery is low and needs to be replaced:

1. Use a T5 TORX screwdriver to loosen the four TORX screws on the back cover of the fob.
2. Remove the back cover of the fob.
3. Remove the old battery.
4. Install a new CR2 type battery.
5. Reinstall the back cover.




The fob is designed to work with a commercially manufactured CR2 battery. Use of another battery may damage the fob and invalidate the warranty.

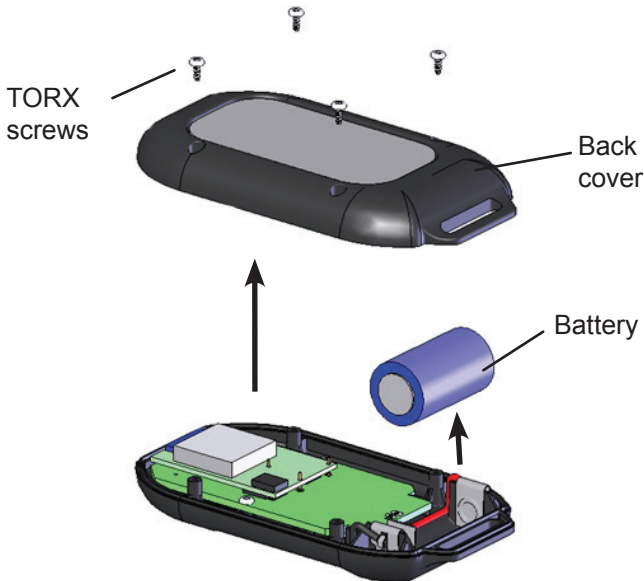

Troubleshooting Guide

Problem	Action
<p>The pump does not vibrate when the on/off button is pressed once and the charger is not connected to the pump.</p>	<ul style="list-style-type: none"> • The pump’s internal battery may not be charged. Charge the pump with the charger for at least 30 minutes and try again. • The on/off button could be damaged. Verify that the rubber boot is intact and hasn’t been torn from impact. Contact your prosthetist if the switch needs to be replaced.
<p>The pump does not begin operation when the on/off button is pressed twice rapidly.</p>	<ul style="list-style-type: none"> • Do not press the on/off button too quickly. Press the button for one second, then release for one second before pressing again, so that the system recognizes separate button presses. • The pump’s internal battery may not be charged. Charge the pump with the charger for at least 30 minutes and try again. • The pump could already be in automatic mode. Pressing twice when the unit is in automatic mode or standby mode will turn the unit completely off. Wait 15 seconds, then press the on/off button once and verify that the pump vibrates to indicate it is in standby mode. After vibration, press a second time to transition the pump to automatic mode.
<p>Pressing the center of the button of the fob does not display the LimbLogic VS logo.</p>	<p>The fob battery may need to be reset. Remove the back cover of the fob per the instructions on page 15, then remove and reinstall the battery. If the problem persists, the battery is likely dead and must be replaced per the instructions on page 15.</p>

Problem	Action
<p>Pump battery does not maintain charge for a full day.</p>	<ul style="list-style-type: none"> • Ensure that the system is properly sealed. • Ensure that the battery has been charged for at least 4 hours.
<p>The fob doesn't communicate with the pump. The LimbLogic VS logo appears and is followed by the No Radio Connection icon.</p> 	<ul style="list-style-type: none"> • Make sure the pump is on by pressing the on/off button once and verify that the pump vibrates in recognition of the button activation. Wait one minute for the fob screen to turn off. Press the fob center button again. • The fob may be too close or too far from the pump. Place the fob about 2 feet away from the pump, and press the fob center button again. • The pump's internal battery may not be charged. Charge the pump with the charger for at least 30 minutes and try again. • Do you have the correct fob-pump combination? The fob and pump are electronically matched during the production process. Ask your prosthetist to contact Ohio Willow Wood with the serial number of the fob and serial number of the pump to verify that the set is matched.
<p>The blue LED light on the charger does not illuminate when plugged into the wall.</p>	<ul style="list-style-type: none"> • The AC electricity to the wall outlet may not be turned on. Try plugging the charger into a different non-switched wall outlet or in another room. • The configurable prong may not be seated properly in the back of the charger's two-pronged adapter. Unplug the charger from the AC outlet. Remove the charger's prong using the sliding latch on the back of the adapter. Verify that the metal contacts are springy and are not completely flat against the adapter. Re-install the plug prongs.

continued on next page

Problem	Action
<p>The green LED on the charger node does not illuminate or blink when the charger is plugged into vacuum pump and AC outlet.</p>	<ul style="list-style-type: none"> • Make sure that the green marks on the charging port and the charging plug are aligned (page 5). Do not force the connection. • The pins on the charger plug may be bent. Unplug the charger from the vacuum pump and inspect the pins. Do NOT try to straighten pins while charger is plugged into an AC outlet! If the pins are bent or broken, contact your prosthetist for repair. • With the charger plugged into an AC outlet and the pump, press and hold the reset button on the charger node for 5 seconds, then release. If the green LED does not blink or turn solid within 5 seconds after the button release, contact your prosthetist for repair. • If charger plug pins have been bent or the plug has been forced into pump, the internal electronics of the pump can be damaged, or the green LED on the charger could be damaged. Use the fob to check the pump battery level, then charge the pump for 2 hours and re-check the battery level to verify that the level has increased. If the level has not increased, contact your prosthetist for repair.
<p>The pump makes a “zip zip zip” sound as it controls the vacuum.</p>	<p>Contact your prosthetist.</p>

Regulatory Information

See www.limblogic.com for detailed WEEE compliance information.

LimbLogic VS Fob



This product contains a user replaceable lithium battery. Replace only with user replaceable CR2 batteries. Discard of discharged batteries in accordance with local regulations. Observe polarity markings on the battery and the fob housing when replacing the battery.



This product complies with IEC 60601-1:1998 +A1:1991 +A2 1995, EN 300 440-1 V1.3.1:2001EMC, EN 301 489-1 V1.7.1:2006, EN 301 489-3 V1.4.1:2002 EMC, and EN55022:1998/A1:2000/A2:2003.



IEC 60601-1 Classifications:

Internally powered

Degree of protection against electrical shock – Type BF Applied Part



In addition to the above compliances, this device complies with EN55022:1998/A1:2000/A2:2003 Class B ITE emissions requirements and all requirements for Low Interference Potential Devices (LIPD) and compliance with the requirements for a General User Licence (GUL).

LimbLogic VS Vacuum Pump



This product contains a rechargeable Lithium Ion Battery. Use ONLY an Ohio Willow Wood charger. For LLV-01042 series Vacuum Pumps use Ohio Willow Wood charger part number LLV-01011. For LLV-01040 series Vacuum Pumps use Ohio Willow Wood charger part number LLV-01010. Use of other devices may result in harm to the device, the user, or both and void the warranty.

Blocking or plugging the exhaust port of the vacuum control assembly will prevent proper operation of the vacuum pump.

The LLVS Vacuum Pump is intended for use on a single patient. Use of the system with multiple patients could lead to cross contamination between patients. Please disinfect systems, or return systems to The Ohio Willow Wood Company to be disinfected, between uses on different patients.

The LLVS vacuum system has been designed for and clinically tested as a suspension aid. It has not been clinically tested for wound healing or other uses. Ohio Willow Wood does not currently support the use of the LLVS for uses other than suspension.

Use only the bolts supplied by The Ohio Willow Wood Company. All testing for the LLVS vacuum system has been tested with the 50mm bolts included with the system. Use of other bolts could result in mechanical failure.



This product complies with IEC 60601-1:1998 +A1:1991 +A2 1995, EN 300 440-1 V1.3.1:2001EMC, EN 301 489-1 V1.7.1:2006, EN 301 489-3 V1.4.1:2002 EMC, EN55022:1998/A1:2000/A2:2003, and CEI/IEC 60529:1989+A1:1999 rating IPX8. (3 meters for 12 hours in fresh water)



IEC 60601-1 Classifications:

Internally powered/Powered from Class II Power Supply

Degree of protection against electrical shock – Type BF Applied Part

This product complies with the requirements of ISO 10328.



In addition to the above compliances, this device complies with EN55022:1998/A1:2000/A2:2003 Class B ITE emissions requirements and all requirements for Low Interference Potential Devices (LIPD) and compliance with the requirements for a General User Licence (GUL).

LimbLogic VS Battery Charger



This product is designed to work with Ohio Willow Wood products specifically designed for its use. For LLV-01042 series Vacuum Pumps use Ohio Willow Wood charger part number LLV-01011. For LLV-01040 series Vacuum Pumps use Ohio Willow Wood charger part number LLV-01010. Use with other products may result in harm to the device, the user, or both and void the warranty. Refer to product manuals to verify compatibility with this charger before attempting to use this charger with the product.

Electrical Ratings:

Rated input: 100 – 240 Vac, 0.6 A, 47 – 63 Hz

Rated output: 5 Vdc, 2.6 A



This product complies with IEC 60601-1:1998 +A1:1991 +A2 1995.

FCC and IC Compliance Information (for the Fob and the Vacuum Pump)

This device complies with Part 15 of the FCC Rules.

Operation is subject to the following two conditions:

1. This device may not cause harmful interference, and
2. This device must accept any interference received, including interference that may cause undesired operation.

Note:

This equipment contains module, FCC ID U3V6221020 (IC 7475A-6221020), which qualifies as an UNLICENSED MODULAR TRANSMITTER per FCC Public Notice DA 00-1407 and RSS-Gen 7.1.2. It has been tested and found to comply with Part 15.249 of the FCC Rules and RSS-210 Annex 2.9. This module has been designed for use by the Ohio Willow Wood Co. This module shall only be designed into products by The Ohio Willow Wood Co.

Warning:

Changes or modifications not expressly approved by The Ohio Willow Wood Company could void the user's authority to operate the equipment.

Required labeling:

Any device incorporating this module must include an external, visible, permanent marking or label which states:

“Contains FCC ID U3V6221020 (IC 7475A-6221020).”

Failure to comply with this requirement will void the user's authority to operate any device that incorporates this module.

The LimbLogic VS is not to be used within the geographical area within a radius of 20 km from the centre of Ny-Ålesund in Norway.

Specifications:

IC	7475A-6221020
FCC ID	U3V6221020
Input voltage	4.5 -12V DC
Input current	37 mA
Operating temperature range	-20°C to +50°C
Frequency range.....	2400 -2483 MHz.
Antenna.....	Permanently soldered to Module
Transmit power	<2mW
Communication type.....	Frequency Hopping Spread Spectrum.
Data rate.....	500 kbps.
Size	1.398” x 0.897”

Electromagnetic Compatibility

Medical electrical equipment needs special precautions regarding electromagnetic compatibility (EMC) and needs to be installed and put into service according to the EMC information provided in this user manual. Portable and mobile radio frequency (RF) communications equipment can affect medical electrical equipment.

Guidance and Manufacturer's Declaration - Electromagnetic Emissions

The LLVS is intended for use in the electromagnetic environments specified below. The customer or the user of the LLVS should assure it is used in such an environment.

Emissions Test	Compliance	Guidance
RF Emissions CISPR 11	Group 2	The LLVS must emit electromagnetic energy in order to perform its intended function. Nearby electronic equipment may be affected.
RF Emissions CISPR 11	Class B	Suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic Emissions IEC 61000-3-2	N/A	
Voltage Fluctuations/ Flicker Emissions IEC 61000-3-3	N/A	

Guidance and Manufacturer's Declaration - Electromagnetic Immunity


The LLVS is intended for use in the electromagnetic environments specified below. The customer or the user of the LLVS should assure it is used in such an environment.

Immunity Test	IEC 60601 Test Level	Compliance Level	Guidance
Electrostatic Discharge IEC 61000-4-2	± 6 kV Contact ± 8 kV Air	± 6 kV Contact ± 8 kV Air	Floors should be wood, concrete, or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical Fast Transient/Burst IEC 61000-4-4	± 2 kV on Power Supply Lines ± 1 kV on Input/ Output Lines	± 2 kV on Power Supply Lines ± 1 kV on Input/ Output Lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV Differential Mode ± 2 kV Common Mode	± 1 kV Differential Mode ± 2 kV Common Mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage Dips, Short Interrupts, & Variations on Power Supply Lines IEC 61000-4-11	< 5% UT (95% dip in UT for 0.5 cycles) < 40% UT (60% dip in UT for 5 cycles) < 70% UT (30% dip in UT for 25 cycles)	< 5% UT (95% dip in UT for 0.5 cycles) < 40% UT (60% dip in UT for 5 cycles) < 70% UT (30% dip in UT for 25 cycles)	Mains power quality should be that of a typical commercial or hospital environment. If the user of the LLVS requires continued operation during mains interruptions, it is recommended that the LLVS be powered from an uninterruptible power supply or a battery.
Power Frequency Magnetic Fields IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE: UT is the a.c. mains voltage prior to application of the test level.

Guidance and Manufacturer's Declaration - Electromagnetic Immunity

The LLVS is intended for use in the electromagnetic environments specified below. The customer or the user of the LLVS should assure it is used in such an environment.

Immunity Test	IEC 60601 Test Level	Compliance Level	Guidance
Conducted RF IEC 61000-4-6 Radiated RF IEC 61000-4-3	3 Vrms 150 kHz to 80 MHz 3 Vrms	3 Vrms 3 V/m	Portable and mobile RF communications equipment should be used no closer to any part of the LLVS, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = 1,2\sqrt{P}$ $d = 1,2\sqrt{P}$ 80 MHz to 800 MHz $d = 2,3\sqrt{P}$ 800 MHz to 2.5 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency range. ^b Interference may occur in the vicinity of equipment marked with the following symbol: 

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and

FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the LLVS is used exceeds the applicable RF compliance level above, the LLVS should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the LLVS.

- b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than [V1] V/m.

Recommended separation distances between portable and mobile RF communications equipment and the LLVS

The LLVS is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the LLVS can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the LLVS as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter	Separation distance according to frequency of transmitter (m)	
	150 kHz to 80 MHz	80 MHz to 800 MHz
	$d = 1,2\sqrt{P}$	$d = 1,2\sqrt{P}$
0.01	0,12	0,12
0.1	0,38	0,38
1	1,2	1,2
10	3,8	3,8
100	12	12

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.



OHIO WILLOW WOOD®

free the body...free the spirit®

15441 Scioto Darby Road

Mt. Sterling, OH 43143

phone 740.869.3377 / 800.848.4930

fax 740.869.4374 www.owwco.com



Ohio Willow Wood Company B.V

Keizersgracht 62/64

1015 CS Amsterdam

The Netherlands



PN-2014-L 17 SEPT 2010

US and Foreign Patents Pending