



LimbLogic™ VS

# Quick-Start Guide

## 1. Turn on the Vacuum Pump.



Press one time

## 2. Start regulating the vacuum.



Press one time

or



Press center button one time

## 3. Information about the system is displayed.

current upper set point (press up and down buttons to adjust)

current vacuum level

Vacuum Pump battery charge level

## 4. To enter Standby Mode:




Press center button until  appears



Standby Mode

## To resume vacuum regulation:




Press center button until  appears

## 5. To adjust the Upper Set Point:



Use up and down buttons to adjust Upper Set Point as desired



To save the change, press center button until  appears

## 6. Turn off the Vacuum Pump.



Press two times



OHIO WILLOW WOOD®

*free the body...free the spirit®*

15441 Scioto Darby Road

Mt. Sterling, OH 43143

phone 740.869.3377 / 800.848.4930

fax 740.869.4374 www.owwco.com



Ohio Willow Wood Company B.V

Keizersgracht 62/64

1015 CS Amsterdam

The Netherlands

