

# *WillowWood*<sup>®</sup>

## Foam Cover Instructions

1. To help prevent tearing the foam when it is tapered into the socket:
  - Hollow out the foam cover to accommodate the socket.
  - Cover the inside of the hollowed-out portion with glue such as Endo or a similar adhesive.
  - Allow to dry.
2. Obtain a rough shape for the foam by using a coarse sanding cone and turning the foam against the direction of the sander.
3. Smooth the foam by using a fine sanding cone and turning the foam in the same direction as the sander.
4. For the Easy-Shape AK Foam Cover, follow the above steps; additionally, hiding the glue seams will be easier if you shape around the circumference of the cover as opposed to up and down the length of the cover.


FOR TECHNICAL INFORMATION, PLEASE CALL  
1-800-848-4930

# *WillowWood*<sup>®</sup>

The Ohio Willow Wood Company  
15441 Scioto Darby Road  
Mt. Sterling, OH 43143  
phone 740.869.3377 / 800.848.4930  
fax 740.869.4374    [www.willowwoodco.com](http://www.willowwoodco.com)



Ohio Willow Wood Company B.V.  
Keizersgracht 62/64  
1015 CS Amsterdam  
The Netherlands

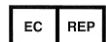

PN-1056-B 2 APR 2014