

# Magnum Foot Instructions

## WHAT'S IN THE BOX

Magnum Foot  
Instructions  
Patient Advisory Warning

## MODIFYING THE FOOT

The urethane foam shell may be lightly sanded with little resulting loss of strength. Do not expose the internal components of the foot, as this could reduce the structural integrity of the foot.

## EXPOSURE TO WATER

The Magnum Foot is essentially water resistant, provided no open cracks have developed and no area of the foot has been sanded. Complete submersion in water is not recommended.

## PATIENT ADVISORY WARNING

The attached Patient Advisory Warning enables you, the prosthetist, to effectively notify your patients of the limitations of the components in their prosthesis, and of the need to monitor their weight and activity levels. Please review the Patient Advisory Warning with the patient upon delivery of a prosthesis with a Magnum Foot. The patient and the prosthetist should then sign the Patient Advisory Warning to acknowledge that it has been reviewed and understood by both parties. Give one signed copy to the patient and place one copy in the patient's file.

If a patient's weight or activity level increases after receiving a prosthesis with a Magnum Foot, the patient should immediately contact the prosthetist to determine whether replacement components are necessary. If a patient continues to use a prosthesis with a Magnum Foot after experiencing an increase in weight and/or activity level, the foot could fail with the possibility of serious injury to the patient.

To ensure that the correct components are selected for each patient, the prosthetist should weigh the patient on scales in the prosthetist's office. Do not rely on the patient's estimate of his/her own weight. Instruct the patient to monitor his/her weight weekly to ensure that it remains in a range appropriate for the prosthetic components being used.

## WARRANTY

The warranty for the Magnum Foot is one year from date of invoice, provided that the foot is selected according to the following criteria:

**Note: The weights listed below are adjusted body weights. Adjusted body weight is defined as the weight of the amputee plus any loads carried by the amputee.**

|             | <u>Level 1</u>   | <u>Level 2</u>     | <u>Level 3</u>     | <u>Level 4</u>     |
|-------------|--|--------------------|--------------------|--------------------|
| Magnum Foot | 350 lb.<br>160 kg.   | 350 lb.<br>160 kg. | 350 lb.<br>160 kg. | 300 lb.<br>135 kg. |
| Level 4:    | Has the ability or potential for prosthetic ambulation that exceeds basic ambulation skills, exhibiting high impact, stress, or energy levels. Typical of the prosthetic demands of the child, active adult, or athlete. |                    |                    |                    |

Use of a Magnum Foot for amputees who do not meet the above criteria or who engage in extremely high and abusive activities is against Ohio Willow Wood's recommendations and will void the one-year warranty. "Extremely high and abusive activities" are defined as activities such as skydiving, karate, and judo; activities that could result in injury to an individual's natural limbs; and activities that expose the prosthesis to corrosives such as salt water.

# WillowWood®

The Ohio Willow Wood Company  
15441 Scioto Darby Road  
Mt. Sterling, OH 43143  
phone 740.869.3377 / 800.848.4930  
fax 740.869.4374 www.willowwoodco.com  
Patent www.willowwoodco.com/education-and-resources/patents



Ohio Willow Wood Company B.V.  
Keizersgracht 62/64  
1015 CS Amsterdam  
The Netherlands



PN-1477-G 19 MAY 2014

**WARRANTY DISCLAIMER**

WillowWood warrants that each product manufactured will, at the time of delivery, be of workmanlike quality and substantially free of defects. WILLOWWOOD MAKES NO OTHER WARRANTY, IMPLIED, OR EXPRESSED, AND MAKES NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. This warranty shall terminate immediately upon an action to combine our products with other materials or in any manner to change the nature of our products. The sole remedy is replacement of the products or credit for the products. WillowWood's liability shall not exceed the purchase price of the product. WillowWood shall not be liable for any indirect, incidental, or consequential damage.

**WILLOWWOOD RETENTION OF RIGHTS**

WillowWood retains all intellectual property rights reflected or incorporated in its physical products, regardless of the transfer of the physical products to another party or parties.