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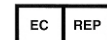
INSTRUCTIONS FOR USE

Note: Do not apply any type of lubricant, lotion, or powder to the residual limb or to the Volume Management Pad or to the Alpha Liner, as these products could damage the Volume Management Pad. The residual limb should be clean, dry, and free of soap residue.

1. Determine where on your limb that you have lost volume. Keep in mind that it is most likely to be a soft tissue area.
2. Select a Volume Management Pad according to the amount of volume that has been lost. For slight volume loss, a small Volume Management Pad can be used. For larger volume loss, a large Volume Management Pad can be used, or several Volume Management Pads can be stacked on top of each other (with the largest pad closest to the skin).

Figure 1 / Figura 1


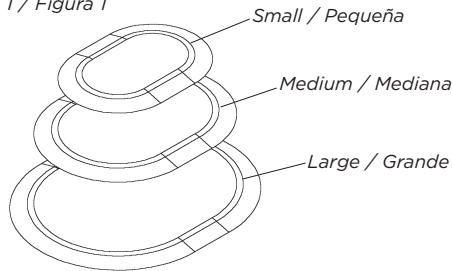
Figure 2 / Figura 2

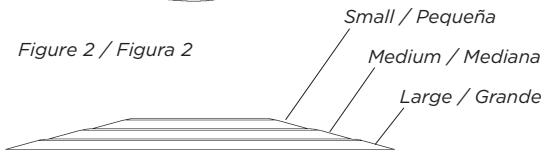Side view of three pads stacked on top of each other.
Vista lateral de las tres almohadillas apiladas.

3. Place the Volume Management Pad(s) on the area of your limb where you have lost volume, with the thin, tapered edge of the pad against the skin and the convex side of the pad to the outside. (Placing the Volume Change Pad upside-down will cause the Pad to change shape.)

Note: The Volume Management Pad will only be effective if it is placed onto the area where volume has been lost. For example, for transtibial amputees,


the area most likely to lose volume is the area at the back of the leg, even though the looseness may be felt on the front of the leg at the tibia.

4. Don the Alpha Liner over your limb according to the instructions provided with the Alpha Liner, then don the prosthesis.

Figure 3 / Figura 3

Placement of the Volume Management Pad at the back of the leg (typical placement for transtibial amputees).

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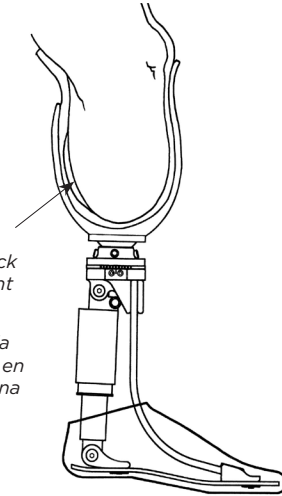Note: If the placement of the Volume Management Pad causes discomfort, or if you have any questions about the use of the Volume Management Pad, please contact your prosthetist for assistance.

5. At the end of the day, remove the Alpha Liner according to the instructions provided with the Alpha Liner, then gently remove the Volume Management Pad for cleaning.

DAILY CLEANING

Each Volume Management Pad should be thoroughly cleaned at the end of each day of wear.

1. Immediately after removing the Volume Management Pad from the limb, clean the Volume Management Pad thoroughly with tap water and a body soap that does not leave a residue. Apply soap either by hand or with a clean, soft cloth or sponge.

Note: Do not scrub the Volume Management Pad. Scrubbing can roughen the surface of the Volume Management Pad, which can then irritate the skin.

2. Take special care to rinse all soap residue from the Volume Management Pad.
3. Place the Volume Management Pad on a soft, lint-free cloth to dry.

WEEKLY CLEANING (FOR DISINFECTION)

1. Place a small amount of ethyl or isopropyl alcohol on a soft, clean cloth.
2. Lightly wipe the Volume Management Pad with the cloth.
3. Place the Volume Management Pad on a soft, lint-free cloth to dry.

Note: Do not soak the Volume Management Pad in the ethyl or isopropyl alcohol. Extended contact with large amounts of ethyl or isopropyl alcohol will stiffen the Volume Management Pad.

STORAGE

When the pads are not in use, store them in the carrying pouch that was provided with the pads.

Alpha Products can melt or burn if exposed to high temperatures or flame. Do not expose your Alpha product to these conditions.

This Alpha product is intended for use on a single patient. Use of the product with multiple patients could lead to cross contamination between patients.

WARNING: To avoid danger of suffocation, keep this product away from babies or children.