

Aluminum Offset Plate Instructions

WHAT'S IN THE BOX

One of the following:

Aluminum 4-Hole 12 mm Offset Plate (FND-135106)

Aluminum Forward Angle Offset Plate (FND-135105)

Aluminum Reverse Angle Offset Plate (FND-135107)

4 M6 x 10 mm Flat Head Capscrews

4 M6 x 16 mm Flat Head Capscrews

Instructions

INSTALLATION

1. Attach the Offset Plate to a 4-hole component using four 6 mm flathead screws of the appropriate length and strength installed through the non-threaded holes on the plate. Apply Loctite 242 (or equivalent) and tighten the screws to 9 ft-lbs (12 Nm).

Note: the Offset Plate can be installed in either an Anterior/Posterior or a Medial/Lateral orientation.

2. Install the mating 4-hole component of your choice onto the Offset Plate using four 6 mm flathead screws of the appropriate length and strength installed through the threaded holes on the plate. Apply Loctite 242 (or equivalent) and tighten the screws to the manufacturer's specifications.

Note: Do not connect any other offset components to the Offset Plate. Doing so would likely increase the stresses on the components and lead to failure of the prosthesis.

WARRANTY

The Warranty for the Aluminum Offset Plate is one year from the date of invoice. A product returned during the warranty period will be evaluated and repaired or replaced at WillowWood's discretion. Use of an Aluminum Offset Plate for amputees whose adjusted body weight is more than 250 pounds (115 kg) or who engage in extremely high and abusive activities is against WillowWood's recommendation and will void the one-year warranty. Adjusted body weight is defined as the weight of the amputee plus any loads normally or routinely carried by the amputee. "Extremely high and abusive activities" are defined as activities such as skydiving, karate, and judo; activities that could result in an injury to an individual's natural foot.

WARRANTY DISCLAIMER

WillowWood warrants that each product manufactured will, at the time of delivery, be of workmanlike quality and substantially free of defects. WILLOWWOOD MAKES NO OTHER WARRANTY, IMPLIED, OR EXPRESSED, AND MAKES NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. This warranty shall terminate immediately upon an action to combine our products with other materials or in any manner to change the nature of our products. The sole remedy is replacement of the products or credit for the products. WillowWood's liability shall not exceed the purchase price of the product. WillowWood shall not be liable for any indirect, incidental, or consequential damage.

WILLOWWOOD RETENTION OF RIGHTS

WillowWood retains all intellectual property rights reflected or incorporated in its physical products, regardless of the transfer of the physical products to another party or parties.

WillowWood®

The Ohio Willow Wood Company
15441 Scioto Darby Road
Mt. Sterling, OH 43143
phone 740.869.3377 / 800.848.4930
fax 740.869.4374 www.willowwoodco.com



Ohio Willow Wood Company B.V.
Keizersgracht 62/64
1015 CS Amsterdam
The Netherlands



PN-1655-E 30 JUL 2013