

Pediatric 1/8" and 1/4" 4-Hole Adapter Spacer Plate Instructions

WHAT'S IN THE BOX

One of the following:

Pediatric 1/8" 4-Hole Adapter Spacer Plate (PPF-994118), includes 4 M5 x 16 Screws

Pediatric 1/4" 4-Hole Adapter Spacer Plate (PPF-994114), includes 4 M5 x 20 Screws

Instructions

Notice: Pediatric components have been tested and approved for use only with other pediatric components. The use of pediatric components with any manufacturer's adult-sized components will void the warranty. The use of pediatric components with another manufacturer's pediatric components is done so at the sole judgment of the attending prosthetist.

INSTALLATION

Attach the Pediatric Spacer Plate to the desired 4-hole component using screws of the appropriate length and strength. Apply Loctite 242 (or equivalent) and tighten the screws to 5 ft-lbs (7 Nm).

Note: if desired, multiple Pediatric Spacer Plates may be installed, as long as the total height of the Spacer Plates does not exceed 3/8" (9.5 mm).

WARRANTY

The warranty for the Pediatric 4-Hole Adapter Spacer Plates is one year from the date of invoice. The use of WillowWood pediatric components with any manufacturer's adult-sized components will void the warranty. Use of a Pediatric 4-Hole Adapter Spacer Plate for amputees whose adjusted body weight is more than 132 pounds (60 kg) or who engage in extremely high and abusive activities is against WillowWood's recommendation and will void the one-year warranty. Adjusted body weight is defined as the weight of the amputee plus any loads normally or routinely carried by the amputee. "Extremely high and abusive activities" are defined as activities such as skydiving, karate, and judo; activities that could result in an injury to an individual's natural foot.

WARRANTY DISCLAIMER

WillowWood warrants that each product manufactured will, at the time of delivery, be of workmanlike quality and substantially free of defects. WILLOWWOOD MAKES NO OTHER WARRANTY, IMPLIED, OR EXPRESSED, AND MAKES NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. This warranty shall terminate immediately upon an action to combine our products with other materials or in any manner to change the nature of our products. The sole remedy is replacement of the products or credit for the products. WillowWood's liability shall not exceed the purchase price of the product. WillowWood shall not be liable for any indirect, incidental, or consequential damage.

WILLOWWOOD RETENTION OF RIGHTS

WillowWood retains all intellectual property rights reflected or incorporated in its physical products, regardless of the transfer of the physical products to another party or parties.

WillowWood™

The Ohio Willow Wood Company
15441 Scioto Darby Road
Mt. Sterling, OH 43143
phone 740.869.3377 / 800.848.4930
fax 740.869.4374 www.willowwoodco.com



Ohio Willow Wood Company B.V.
Keizersgracht 62/64
1015 CS Amsterdam
The Netherlands



PN-1668-D 14 AUG 2012