

Replacing the Pathfinder® II Valve Assembly

Important Note: Before removing the Valve Assembly, release all air pressure from the Pneumatic Heel Spring.

Using a 3/8" thin-walled socket wrench:

1. Remove the old Valve Assembly.
2. Install the replacement Valve Assembly.
3. Tighten the replacement Valve Assembly to 50 inch-pounds.

WillowWood®

The Ohio Willow Wood Company
15441 Scioto Darby Road
Mt. Sterling, OH 43143
phone 740.869.3377 / 800.848.4930
fax 740.869.4374 www.willowwoodco.com



Ohio Willow Wood Company B.V.
Keizersgracht 62/64
1015 CS Amsterdam
The Netherlands

