

# Titanium Foot Pyramid/Double-Ended Pyramid Instructions

## WHAT'S IN THE BOX

One of the following:

Titanium Foot Pyramid (2LTC-1208)

Titanium Double-Ended Pyramid (MHD-DE-P)

Instructions

## TITANIUM DOUBLE-ENDED PYRAMID INSTALLATION

Install the mating pyramid receivers of your choice onto each end of the Double-Ended Pyramid. Once alignment has been achieved, follow the torque and threadlocker specifications provided by the manufacturer of the pyramid receivers.

## TITANIUM FOOT PYRAMID INSTALLATION

1. Place the Titanium Foot Pyramid onto the foot:
  - If using the Impulse® Foot or Carbon Copy 2® Foot, place the Titanium Foot Pyramid so that the ridge on the distal surface of the pyramid fits into the slot on the proximal surface of the foot.
  - If using WillowWood's SACH Foot, the ridge on the bottom of the Titanium Foot Pyramid will grip into the composite keel when it is tightened. If you will be changing the position of the adapter numerous times (during a temporary alignment, for example), we recommend that you use an Interface Plate (Part No. 3F-6CC-002), a leather interface, or some similar temporary device to prevent excessive deformation of the keel. WillowWood will not accept returns of feet with deformed keels.
2. Attach the Titanium Foot Pyramid using the foot bolt that was supplied with the foot. Apply Loctite 242 (or equivalent) to the foot bolt and tighten the bolt to 30 ft-lbs (41 Nm).

### Notes:

- **If using another manufacturer's foot, be sure to check for adequate foot bolt thread engagement.**
- **Do not modify the Titanium Foot Pyramid in any way.**

## WARRANTY

The warranty for the Titanium Foot Pyramid and Titanium Double-Ended Pyramid is one year from date of invoice. Use of a Titanium Foot Pyramid or Double-Ended Pyramid for amputees whose adjusted body weight is greater than 350 lbs (160 kg) or who engage in extremely high and abusive activities is against WillowWood's recommendations and will void the one year warranty. Adjusted body weight is defined as the weight of the amputee plus any loads normally or routinely carried by the amputee. "Extremely high and abusive activities" are defined as activities such as skydiving, karate, and judo; activities that could result in injury to an individual's natural limbs; and activities that expose the prosthesis to corrosives such as salt water.

## WARRANTY DISCLAIMER

WillowWood warrants that each product manufactured will, at the time of delivery, be of workmanlike quality and substantially free of defects. WILLOWWOOD MAKES NO OTHER WARRANTY, IMPLIED, OR EXPRESSED, AND MAKES NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. This warranty shall terminate immediately upon an action to combine our products with other materials or in any manner to change the nature of our products. The sole remedy is replacement of the products or credit for the products. WillowWood's liability shall not exceed the purchase price of the product. WillowWood shall not be liable for any indirect, incidental, or consequential damage.

## WILLOWWOOD RETENTION OF RIGHTS

WillowWood retains all intellectual property rights reflected or incorporated in its physical products, regardless of the transfer of the physical products to another party or parties.

# WillowWood®

The Ohio Willow Wood Company  
15441 Scioto Darby Road  
Mt. Sterling, OH 43143  
phone 740.869.3377 / 800.848.4930  
fax 740.869.4374 www.willowwoodco.com



Ohio Willow Wood Company B.V.  
Keizersgracht 62/64  
1015 CS Amsterdam  
The Netherlands



PN-1805-F 09 AUG 2012