


Replacing the Carbon Heel on the DuraLite™ Foot

1. Using a 5 mm hex wrench, remove the two M8 cap screws as shown.
2. Remove the heel.
3. Insert the new heel.
4. Apply Loctite 242 (or equivalent) to the M8 cap screws.
5. Install the M8 hardened flat washers onto the M8 cap screws.
6. Install the screws with the washers. Tighten the screws to 16 ft-lbs (21 Nm).

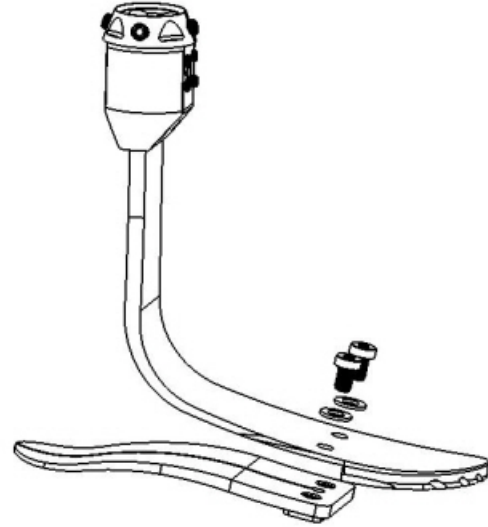

WillowWood®

The Ohio Willow Wood Company
15441 Scioto Darby Road
Mt. Sterling, OH 43143
phone 740.869.3377 / 800.848.4930
fax 740.869.4374 www.willowwoodco.com



Ohio Willow Wood Company B.V.
Keizersgracht 62/64
1015 CS Amsterdam
The Netherlands



PN-2037-B 06 NOV 2012