

# Titanium Rotating Slide Adapter Instructions

## WHAT'S IN THE BOX

Titanium Rotating Slide Adapter (FND-224150)  
4 M6 X 16 Dacromet Screws  
4 M6 X 35 Dacromet Screws  
Instructions

## INSTALLATION

1. Using the four M6 X 16 screws included with the Slide Adapter, attach the Slide Adapter to the socket. (Alternately, use the M6 X 35 screws that are also included with the Slide Adapter to attach the Slide Adapter to an Alpha Lock or Alpha Locking Lanyard.) Tighten the screws to 9 ft-lbs (12 Nm)
2. Connect a pyramid receiver component of your choice to the Slide Adapter. Tighten the setscrews using the torque and threadlocker specifications provided by the manufacturer of the pyramid receiver.
3. To make a slide adjustment:
  - a. Loosen two adjacent 8 mm setscrews on the pyramid receptacle.
  - b. Slide the pyramid to the desired position.
  - c. Tighten the 8 mm setscrews using the torque and threadlocker specifications provided by the manufacturer of the pyramid receptacle.
4. To make a rotation adjustment:
  - a. Loosen two adjacent 8 mm setscrews on the pyramid receptacle.
  - b. Rotate the pyramid to the desired position.
  - c. Tighten the 8 mm setscrews using the torque and threadlocker specifications provided by the manufacturer of the pyramid receptacle.

## WARRANTY

The warranty for the Titanium Rotating Slide Adapter is one year from date of invoice. Use of a Titanium Rotating Slide Adapter for amputees whose adjusted body weight is greater than 350 lbs (160 kg) or who engage in extremely high and abusive activities is against WillowWood's recommendations and will void the one-year warranty. Adjusted body weight is defined as the weight of the amputee plus any loads normally or routinely carried by the amputee. "Extremely high and abusive activities" are defined as activities such as skydiving, karate, and judo; activities that could result in injury to an individual's natural limbs; and activities that expose the prosthesis to corrosives such as salt water.

## WARRANTY DISCLAIMER

WillowWood warrants that each product manufactured will, at the time of delivery, be of workmanlike quality and substantially free of defects. WILLOWWOOD MAKES NO OTHER WARRANTY, IMPLIED, OR EXPRESSED, AND MAKES NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. This warranty shall terminate immediately upon an action to combine our products with other materials or in any manner to change the nature of our products. The sole remedy is replacement of the products or credit for the products. WillowWood's liability shall not exceed the purchase price of the product. WillowWood shall not be liable for any indirect, incidental, or consequential damage.

## WILLOWWOOD RETENTION OF RIGHTS


WillowWood retains all intellectual property rights reflected or incorporated in its physical products, regardless of the transfer of the physical products to another party or parties.

# WillowWood®

The Ohio Willow Wood Company  
15441 Scioto Darby Road  
Mt. Sterling, OH 43143  
phone 740.869.3377 / 800.848.4930  
fax 740.869.4374 [www.willowwoodco.com](http://www.willowwoodco.com)



Ohio Willow Wood Company B.V  
Keizersgracht 62/64  
1015 CS Amsterdam  
The Netherlands

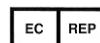

PN-2341-A 16 JUN 2014