

If you do not understand these warnings, please discuss them with your prosthetist.



Caution: Alpha products can melt or burn if exposed to high temperatures or flame. Do not expose your Alpha product to these conditions.



Caution: This Alpha product is intended for use on a single patient. Use of the product with multiple patients could lead to cross contamination between patients.



Warning: To avoid danger of suffocation, keep this product away from babies or children.



Warning: WillowWood One® Transtibial System components have been designed and tested for use only with other WillowWood One Transtibial System components. Use with other sealing systems may result in loss of suspension resulting in injury.

WillowWood®

One Sleeve User Instructions



Please contact your prosthetist if you have any questions about the use of the One Sleeve.

WillowWood®

The Ohio Willow Wood Company
15441 Scioto Darby Road
Mt. Sterling, OH 43143
phone 740.869.3377 / 800.848.4930
fax 740.869.4374 www.willowwoodco.com



Ohio Willow Wood Company B.V.
Keizersgracht 62/64
1015 CS Amsterdam
The Netherlands



PN-2511-B 07 APRIL 2017

PUTTING ON THE SLEEVE

1. Fold the large end of the sleeve down onto the socket to expose the open end of the socket.



2. Step into the socket and pull the sleeve up onto the residual limb.



STORING

When the prosthesis is not being worn, the bottom portion of the sleeve should be left on the socket.

The top half of the sleeve should be stored as shown below.

The top half of the sleeve should be unrolled and allowed to drape into socket for protection.

CORRECT



It is also acceptable to allow the sleeve to extend out of the socket.


CORRECT



Do not leave the top half of the sleeve rolled back onto the socket edge.

The sleeve could be damaged if the prosthesis falls over.

INCORRECT

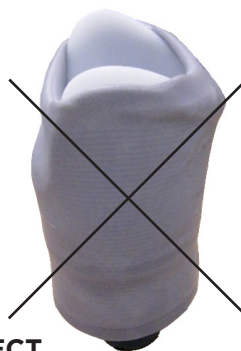

CLEANING

Gently clean the sleeve with soap and water. There is no need to remove the sleeve from the prosthesis.

DISINFECTING

If the silicone part of the One Sleeve touches your skin, it will need to be disinfected. Once a week, disinfect the area of the silicone that touches the skin.

1. Place a small amount of ethyl or isopropyl alcohol on a soft, clean cloth.
2. Lightly wipe the silicone with the cloth.
3. Rinse off the alcohol with water.

Note: Do not soak the sleeve in the ethyl or isopropyl alcohol. Soaking the sleeve in alcohol will stiffen the sleeve.