

OMEGA[®] Software

Version 1.6.3 Release Notes

OMEGA Software, version 1.6.3 is an update released in **September 2018**. To contact WillowWood for advice or help, select **Help** from the Main Menu Bar of the software. Additional OMEGA product information can be found in the [OMEGA section of willowwoodco.com](#).

1. ADDITIONS

- In the **Measurements** pane for AFOs: **Height** is now measured from the bottom of the foot up the center line to the most proximal point. **Length** is now measured from the back of the heel through the center line to the toe that extends the farthest.
- To reduce issues caused by attempting to send large files, scan files will be compressed if necessary (without losing the shape) so that the maximum file size will be 20 MB.

2. REMOVALS

No functions were removed in this release.

3. CHANGES

- The **Notes** field associated with patient shape files will now accept 500 characters.
- In the Fab Stack, the list of available **Ship To** locations will now be displayed in a pop-up window.
- In the Design Stack for the Alpha DESIGN[®] Liner, the **Disarticulation Buildup** step now happens after the **Posterior/Medial Thickness** step, and a **Preserve Posterior/Medial** checkbox has been added, so that adjustments made in the **Posterior/Medial Thickness** step will be retained.

4. FIXES


- The **Change Length** tool now preserves the shape of the distal end.
- In the **Measurement** pane, **Volume** is now being calculated correctly regardless of which units are selected.
- In **Comparison Reports**, the Primary Save Date and the Comparison Save Date are no longer the same; Primary Save Date is the date and time of the last time the current model was saved, and Comparison Save Date is the date and time of the last time the comparison model was saved.
- In the **Views** pane, when a file is saved with the **Compare** option set to **On**, that option will remain on when the saved file is opened again, instead of reverting back to **Off**.
- In the **Patient Details** step of the Design Stack for the Alpha DESIGN Liner:
 - The circumference and length fields will be updated if the user makes changes to these measurements in the Capture Stack.
 - OMEGA now recognizes that 30.5 cm and 305 mm are equal to 12 inches.
- The **Notes** field in the Fab Stack will no longer default to displaying the notes entered for the previous order.
- In the **Posterior/Medial Thickness** step of the Design Stack, a ridge is no longer displayed where the gel is thickest.
- When the goniometer is moved in the **Flex/Extend** step in the Design Stack, the model now moves in the same direction as the goniometer.

WillowWood[®]

The Ohio Willow Wood Company
15441 Scioto Darby Road
Mt. Sterling, OH 43143
phone 740.869.3377 / 800.848.4930
fax 740.869.4374 www.willowwoodco.com



Ohio Willow Wood Company B.V.
Keizersgracht 62/64
1015 CS Amsterdam
The Netherlands
Patent www.willowwoodco.com/education-and-resources/patents

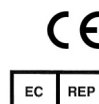

PN-2556-E 21 SEPT 2018