

# OMEGA<sup>®</sup> Software

## Version 1.6.4 Release Notes

OMEGA Software, version 1.6.4 is an update released in **November 2018**. To contact WillowWood for advice or help, select **Help** from the Main Menu Bar of the software. Additional OMEGA product information can be found in the [OMEGA section of willowwoodco.com](#).

### 1. ADDITIONS

- In the Design Stack for the Alpha DESIGN<sup>®</sup> Liner, the user may now insert an additional **Add Gel** step by clicking the gear icon in the existing **Add Gel** step.

### 2. REMOVALS

No functions were removed in this release.

### 3. CHANGES

- For AFOs, indicating the **Side of 1st MT Head** is now done in the the **Final Alignment** step instead of in the **Landmarks** step.

### 4. FIXES


- For transtibial Comparison Reports that include a PTB landmark, and for transfemoral Comparison Reports that include an Ischium landmark, the measurements will now be displayed on the correct row of the table.
- The Views and Measurements pane is now displayed upon opening any saved file.
- In the Landmarks Manager, landmarks that are listed in the **Current Landmarks** list will not also be included in the **Available Landmarks** list.
- The option to hide targets on imported STL/OBJ files or scanned files (by clicking the gear icon in the Landmark step in the Capture Stack and selecting **Hide scan targets**) is now also available for AOP files.
- Landmarks can no longer be edited or moved in the Design Stack.
- In **Settings**, the user can now enter a **Billing Address** for multiple offices even if **Same as Shipping** was checked for the first office.
- In the **Add Material** step of the Design Stack, clicking the gear icon and selecting **Clear changes** will now correctly restore the default shape.
- If a landmark is selected as a custom region in the **Add Material** step of the Design Stack, the selected landmark will be displayed in white if the Landmark option in the **Views** panel is set to **On**; otherwise the landmark will not be displayed in white.
- For Alpha DESIGN Liners and sockets, any changes made to the shape after the **Import/Scan** step will be cleared if the user goes back to the **Import/Scan** step to change the alignment.
- In the **Landmark** step of the Capture Stack, clicking the gear icon and selecting the **Clear changes** option will now correctly undo the selection of a landmark.
- In the Fab Stack, the scroll buttons are now enabled at all times, regardless of screen resolution, to allow viewing of all of the order information.

# WillowWood<sup>®</sup>

The Ohio Willow Wood Company  
15441 Scioto Darby Road  
Mt. Sterling, OH 43143  
phone 740.869.3377 / 800.848.4930  
fax 740.869.4374 [www.willowwoodco.com](http://www.willowwoodco.com)



Ohio Willow Wood Company B.V.  
Keizersgracht 62/64  
1015 CS Amsterdam  
The Netherlands  
Patent [www.willowwoodco.com/education-and-resources/patents](http://www.willowwoodco.com/education-and-resources/patents)

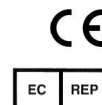

PN-2556-E 19 NOV 2018