

OMEGA[®] Software

Version 1.7.1 Release Notes

OMEGA Software version 1.7.1 is an update released in **June 2019**. To contact WillowWood for advice or help, select **Help** from the Main Menu Bar of the software. Additional OMEGA product information can be found in the [OMEGA section of willowwoodco.com](http://www.willowwoodco.com).

The following changes all apply to tools used for creating an Alpha DESIGN[®] Liner.

1. ADDITIONS

- When using the **Disarticulation Buildup** tool, users will now have the ability to **Preserve Anterior** thickness.
- When the **Warning! Gel thickness will be limited to 25 mm** message is displayed, the user will now have the option to select **Don't show again** to prevent further display of the message.
- Accent marks on vowels are now accepted when entering text.

2. REMOVALS

- No functions were removed in this release.

3. CHANGES

- The maximum gel thickness that is allowed with the **Distal Thickness** tool has been increased from 15 mm to 25 mm.
- The **Preserve Posterior** and **Preserve Medial** options now preserve the thickness for the entire posterior side (for TT shapes) or medial side (for TF shapes). Previously, these options preserved the thickness from the distal red ring to the proximal end.

4. FIXES

- The file size no longer increases significantly when the **Add Gel** or **Remove Gel** tool is used.

WillowWood[®]

WillowWood Global LLC
15441 Scioto Darby Road
Mt. Sterling, OH 43143
phone 740.869.3377 / 800.848.4930
fax 740.869.4374 www.willowwoodco.com



Ohio Willow Wood Company B.V.
Keizersgracht 62/64
1015 CS Amsterdam
The Netherlands
Patent www.willowwoodco.com/education-and-resources/patents

